



Donning and Doffing Procedure

Infectious Control Ensemble: Doffing

1. Disinfect your gloves using hand sanitizer, bleach spray or disinfectant wipes. (Allow to air dry)
2. Remove the shroud:
Hold mask with one hand while pulling the shroud from the back and towards the front with the other hand over P100 canister and tubing. Repeat step-1 and disinfect your gloves.
3. Unseal the adhesive flap and unzip the suit.
(If using Tian suit, peel the ChemTape carefully prior to unzipping.)
4. Remove the hood by grabbing it from the top and pulling it toward the back.
5. Remove the ChemTape from your wrists.
6. Grasp the fingertips of the outer gloves in order to loosen the gloves.
7. Grab the sleeve and outer glove together and pull your arm out the suit. Keep hand over chest to avoid contact with the outside of suit.
8. Slide hand from the chest to the opposite side and from the inside of the suit, slide the sleeve off, only removing the outer glove.
9. Keeping hands inside the suit, roll the suit down to the ankles. (Repeat step-1 before rolling down suit)
10. Sitting on a smooth surface, remove feet from booties and suit together from the inside.
11. Disinfect your inner gloves by repeating step-1. Add a new fresh pair of gloves over the inner gloves.
12. Carefully remove the APR or SCBA facepiece.
(Dispose of cannister in the red bag. APR or SCBA shall be placed in a clear bag for later decontamination.) Disinfect or wash hands.



Important Points

- If highly suspected Ebola patient encountered, HazMat will perform "Valet Doffing".
- Inspect the suit. If there is minimal contamination, wipe off and proceed with doffing if directed by Officer.
- You must have a large clear bag and bio-hazard bag ready.
- WASH OR DISINFECT HANDS IMMEDIATELY AFTER DOFFING

