



Donning and Doffing Procedure

Infectious Control Ensemble: Donning

1. Insert feet into legs of suit. Sitting on a smooth surface, don the rubber boot covers and pull up suit to waist.
2. Don your first pair of Nitrile gloves.
3. Slip your arms into the suit.
4. Don your APR or SCBA facepiece.
5. Put on the hood then zip up the suit.
6. Seal the adhesive flap over the zipper of the suit. (When using the Tian suit, use ChemTape to seal flap)
7. Don the second pair of gloves ensuring the cuff of the glove is over the sleeve of the suit. Using ChemTape, seal the outer glove cuff to the sleeve of the suit. Make sure there is a tab at the end of the tape to allow for easy doffing.
8. Put on the shroud/hood over the hood. Ensure that the elastic band of the shroud/hood is properly fitted around the APR or SCBA so that no skin is exposed.
9. Inspect and attach the P100 canister to your APR or SCBA facepiece.



Important Points

- Avoid or remove any sharp objects as to not tear the suit or rubber boots.
- Inspect the suit before donning.
- Keep your hands away from your face.
- Inspect the APR at the start of each tour.
- Limit surfaces touched.
- Avoid stepping in or soiling the suit with the patient's bodily fluids if at all possible.
- Remember to change gloves as often as necessary