


9

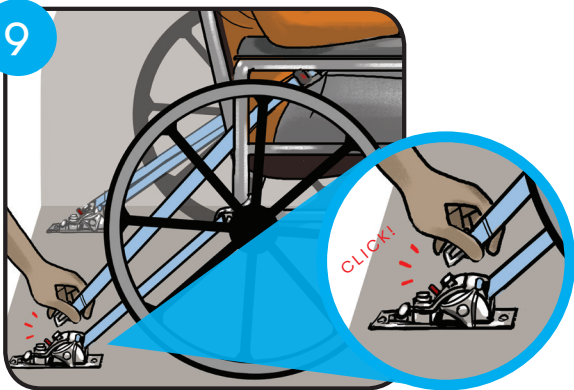

Connect lap belt to back securements.


10

Are you comfortable?



Tighten the lap belt.

11

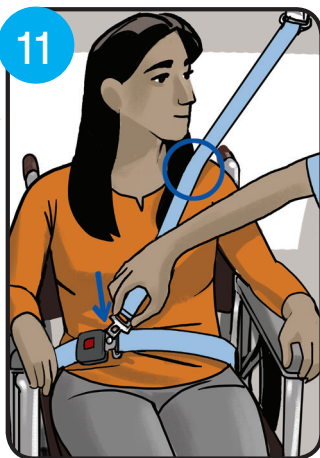


Connect shoulder strap.

12



Tighten the shoulder strap.

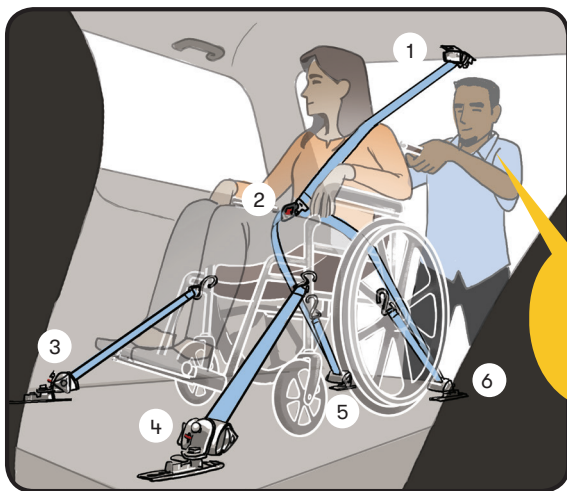

Test the securements - gently push and pull the wheelchair to make sure the securements keep the passenger in place.

I am going to push you back and forth gently to make sure you are secured.

YOU'RE READY TO RIDE!




WHEELCHAIR ACCESSIBLE VEHICLE HOW-TO

Helpful Tips

Don't rush!
Take your time.

**Use all six securement
straps. Do not use
broken equipment.**

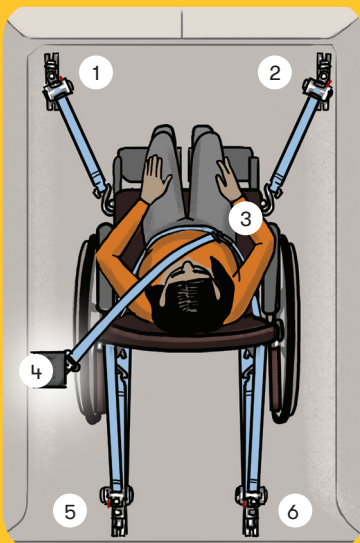

I'm going to secure
the shoulder belt to
the lap belt...



**Tell your
passengers
what you are
doing.**

Can you make this lap
belt a little looser?




**Listen
to your
passengers.**

**Keep your
vehicle clean.**




ONBOARDING

1

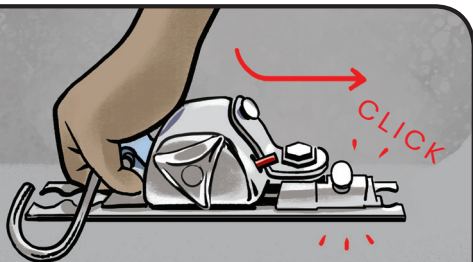

Slide front securement into tracks.

2

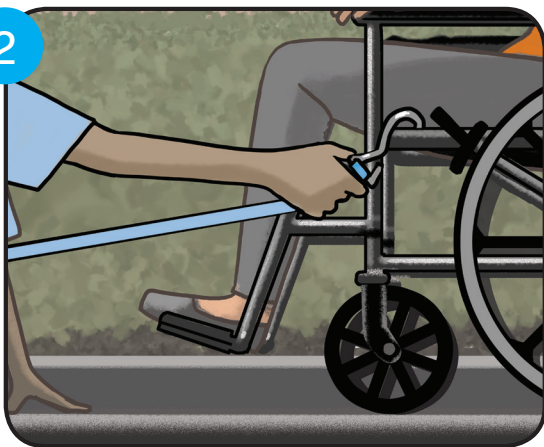

Attach hook to securement spot.


3




Assist passenger into vehicle.

Can I help you into the vehicle?


4

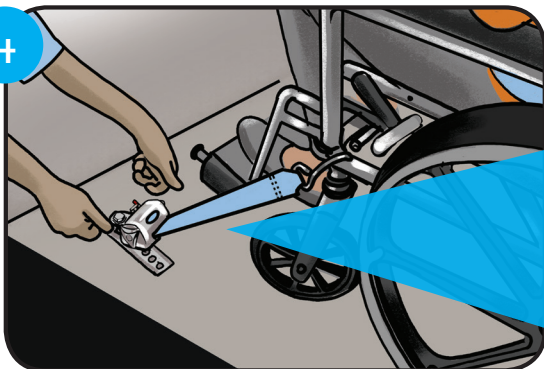

Tighten front securements.

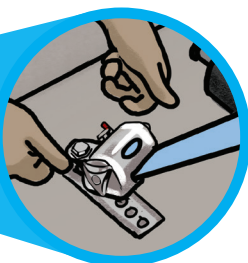


5

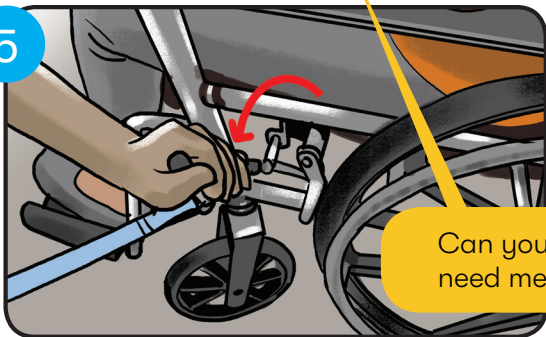

Lock the brakes.


Can you lock your brakes, or do you need me to?

6

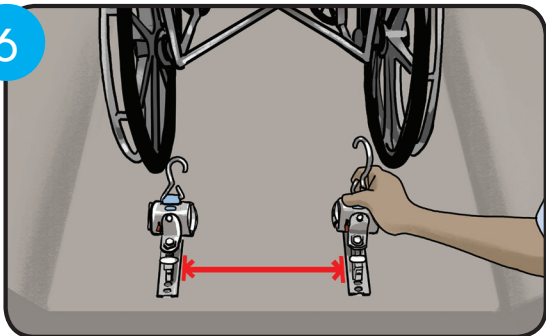


**Slide back
securments into
tracks.**

7

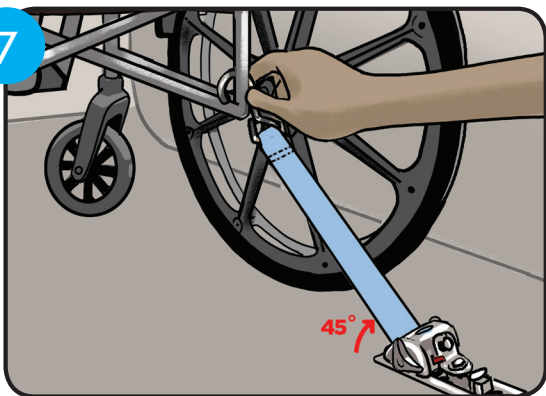

**Hook straps onto
back of wheelchair
and tighten.**

8

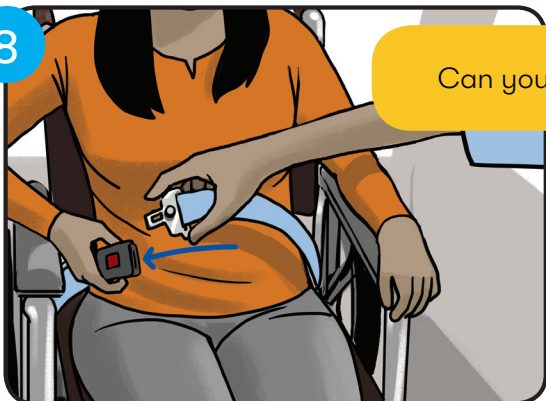

Can you hold this in place for me?

**Place lap belt across
passenger's lap. The
red button should be
at right hip.**